

APS-200C

APS200Duo on CAR-ACC-MC-200 Cart (Mobile)
CAR-ACC-SC-200 Cart (Stationary)

FEATURES

DUAL HIGH-DEFINITION CAMERAS

The remote clinician can switch between and control the positions of both cameras

MICROPHONE ARRAY

The microphone array picks up the voices at the bedside. The remote clinician controls the mic volume

INFRARED LIGHT

The infrared light allows the remote clinician to see clearly in poorly lit rooms without turning on lights that would disturb a patient.

MONITOR

The monitor displays the remote clinician's video when in a two-way call and also shows the iObserver indicator.

SPEAKER

The speaker bar is where the bedside hears the audio on their side from the remote clinicians, who controls the volume.

CLEANING THE CART BETWEEN PATIENT USES:

1. Before cleaning:
 - a. Turn off and unplug the cart
 - b. Do not grab the camera or move the camera head
 - c. Do not spray directly on the camera lenses, camera, speaker/microphone
 - d. Do not unplug or move any connected wires to the monitor, battery, or APS200 DUO while cleaning
2. Clean the cart chassis with hospital approved cleaning wipes (bleach wipes if exposed to c-diff). Avoid excess dampness and do not allow liquids to spill inside any of the devices
3. Clean the APS200 using a dry cloth or duster, use medical wipes sparingly, rubbing lightly on the enclosure. (Do not use bleach) Lightly wipe alcohol wipes on camera lens if needed.
4. Clean the monitor with "Whoosh" or PDI Cleaners. Do not use ammonia

CART SET-UP CONSIDERATIONS

- Place the cart 3-5 feet from the foot of the patient's bed
- Avoid placing the wires/cords or cart in the walkway to avoid tripping hazards
- Ensure cart is plugged into its power outlet for stationary carts or whenever possible for mobile carts
- Clean the cart according to hospital policy before placing in another patient's room – see policy
- Never touch, cover or move the camera to reposition, as it will break the camera – move the cart instead.
- Only move the cart by using the handle, never move by pulling on the monitor.



FEATURES AND SET-UP




FEATURES

- 1 Powered Speakers:** Two build in speaker, behind wipeable grill. Dedicated for Caregility Audio
- 2 PTZ Camera:** 40x Zoom
- 3 5x Digital Zoom Camera with Lens Cover**
- 4 Control:** Switching, Far End Pan/Tilt/Zoom Control
- 5 6-Microphone Array:** Auto-Gain and Noise Cancelling to isolate voices in noisy room environments
- 6 Dual Camera IR light:** Provides light for Nightvision in dark and low-light conditions
- 7 Indicator Lamp:** Programmable LED shows On/Off, In Call/Use, Connectivity Error

CART SET-UP CHECKLIST

1. Notify remote clinician when the cart is placed in patient room
2. Remote clinician's camera is in a 2-way audio /video session to verify
3. On initiation with APS200MC carts with battery packs, if the blue light is not illuminated on the battery pack, HOLD POWER BUTTON for 2-3 seconds to power on.
 - ☐ Speaker /microphone is on high volume and not muted
 - ☐ Video monitoring sign is in place on door or wall
 - ☐ Patient has been educated and documentation has been placed in chart
 - ☐ Bedside and remote clinician validate patient identity
 - ☐ Set "Home" position
 - ☐ Bedside can hear remote clinician
 - ☐ Remote clinician can hear bedside
 - ☐ Bedside can see remote clinician
 - ☐ Remote clinician can see bedside
 - ☐ Doorbell is heard

POWER AND BATTERY LOCATIONS

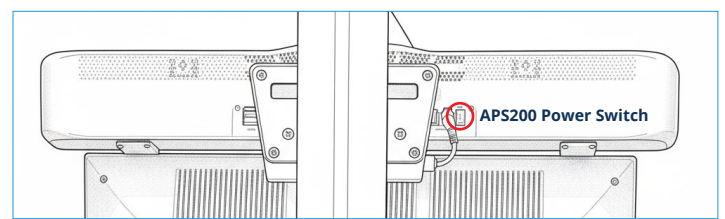

(figure 1)

APS200 Power Switch: The location is on the back of the unit.



(figure 2)



(figure 3)

Battery Docking: Located in front above the wheel base (figure 2), there is a release button on the top and there can be a side locking screw on the left side of the handle. This requires the provided security bit.

Battery: A 289Wh lithium-Ion battery with power indicator (figure 3)

APS Battery Pack: Only Applies to CAR-ACC-MC-200 Carts (figures 2-3)